

High Volume Tap (30-40 cc)		Diagnostic Code:
Opening/Closing Pressures		
GaitRite Testing pre-LP		
GaitRite Testing post-LP		

Tube Number	Tube Type	Volume (cc)	Test Needed	Tests and Codes	Requirements	Test Form (Lab Tag)	Test Location	Comments
1	LP tray #1 Routine Tap	3.0		Protein (CTP), Glucose (CGL) and Lactic Acid (CLA)	Refrigerate and Send with Serum	Transplant Immun/Flow Cytometry	WBLM	Send with 5cc Whole Blood (Tiger Top Tube) (GLUR)
2	LP tray #2 Routine Tap	4.0		Micro (CSFC, WFUNC, VCULT, AFCUL)	Sterility Essential, Room Temperature	Micro	HFH- Micro	CSF Gram Stain (AGMST)-performed at WBLM
3	LP tray #3 Routine Tap	5.0		Cytology	Refrigerate	Cyto-pathology	HFH	
4	LP tray #4 Routine Tap	1.0		Cell Count, Diff (CSFCD)	Refrigerate	Transplant Immun/Flow Cytometry	WBLM	
5	extra	2.0		VDRL (VDCSF), Cryptococcus (AGCRY)	Refrigerate	VDRL- Trans Immun/Flow AGCRY-Micro	HFH- Micro	
6	extra	1.0		ACE (CACE2)	Refrigerate	Transplant Immun/Flow Cytometry	Send out	
7	extra	2.0		Lyme (CLYME)	Refrigerate	Transplant Immun/Flow Cytometry	Send out	
8	extra	1.0		IgG/Oligoclonal bands (OBEV)	Refrigerate and Send with Serum	Multi	HFH- Chem	Send with 5cc Whole Blood (Tiger Top Tube)
9	extra	1.0		Flow Cytometry (RTHLA; LLY)	Room Temperature	Transplant Immun/Flow Cytometry	HFH	
10	polypropylene	5.0		JCF 14 3 3 CSF Prion (P1433)	Freeze	Universal	Send out	Urine 100cc Freeze- ship with CSF to National Prion Institute
11	polypropylene	5.0		CSF- Freeze (SFS)	Freeze	Universal	WBLM	
12	polypropylene	1.0		West Nile (CWNV)	Freeze	Universal	Send out	
13	polypropylene	3.0		Tau/Abeta	RN pick-up/call contact #below	Athena	Athena	Handle as "STAT"; Freeze if Overnight
14	extra	2.0		Epstein Barr (EBVPCR)	Refrigerate	Micro write in	Send out	
15	extra	1.0		Varicella Zoster (VZVPCR)	Sterility Essential	Micro	Send out	
16	extra	1.0		Herpes simplex (HSVPCR)	Sterility Essential	Micro	HFH	
17	extra	1.0		Cytomegalovirus (CMVPCR)	Sterility Essential	Micro	HFH	
18								

Ordering Physician		Tests #1 and #8	Contact:
Physician Code		Send CSF with 5cc whole blood for each test in two Tiger Top tubes	*All tubes must be singly wrapped, sterile
Location			#1- #4 Routine Tap