


THERE IS ONLY **1** WAY TO PROPERLY LABEL A VACUTAINER TUBE.

GET IT STRAIGHT OR RESULTS WILL WAIT

LABELING INSTRUCTIONS

Holding specimen vertical, firmly and cleanly wrap the barcode label around the vacutainer tube (as shown in the image). Whether label stock has a perforated edge or not, patient name is always right side vertical with accession number positioned at the top.

If possible, leave a 'window' to view tube contents.

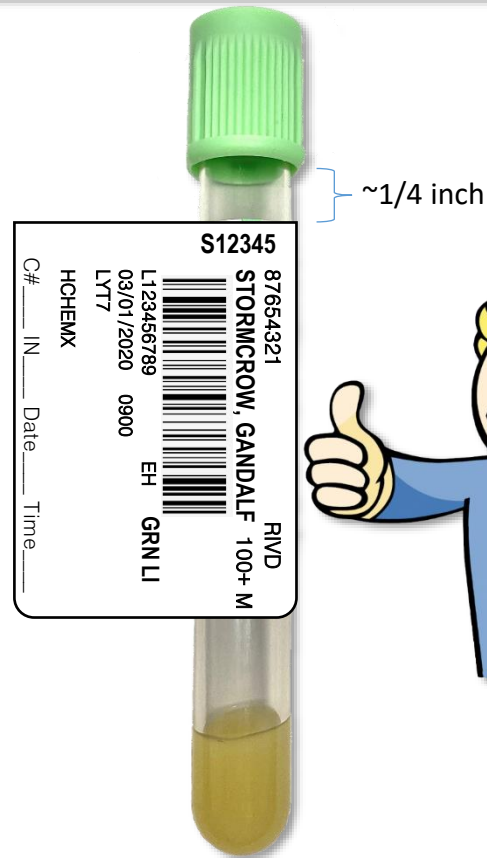

* For print quality issues, contact the Help Desk at 248.853.4900.

CREATIVE LABELING RESULTS IN DELAYS

NO BARBER POLE

NO UPSIDE DOWN

INCORRECT ORIENTATION 1

INCORRECT ORIENTATION 2

Only the primary Sunquest barcode is relevant. Cover any other exposed barcode or it will interfere with analyzer barcode readers.

Labels with pitted or poorly printed barcodes are not acceptable.



TOO LOW

TOO HIGH

NO DOUBLE LABELS

NO WRINKLES!